
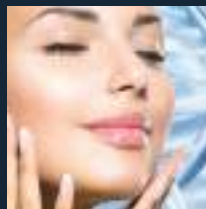


*From fairest creatures we desire increase, That thereby
beauty's rose might never die, But as the ripper should*

mya^{6.0}
Led mask

HPL
AIRYe⁺

THE BEAUTY CULT, ART IN SLIMMING.

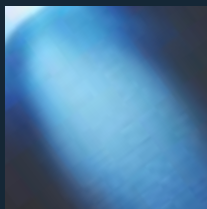


C E S A R E
Q U A R A N T A


Made in Italy

*But as the ripper should by time decease, His tender heart
might bear his memory; but thou, contracted to thine
own bright eyes*



Certified beauty

Cesare Quaranta Quality

Cesare Quaranta srl is an ISO 9001:2015 certified company for projecting, manufacturing and technical servicing devices for aesthetics.

All devices produced comply with Cesare Quaranta employs a quality management system according to ISO 13485:2016 for the design, production and technical assistance of electro-medical devices for dermocosmetics and in according to ISO 9001: 2015 for the design, production and technical assistance of equipment for aesthetic and medical use with the aim of consolidating the quality level achieved in the production processes and reinforcing the renewal and the continuous improvement already present in all the company areas.

Quality management of equipment technical assistance of the protects the customer from the point of view of short timing and excellent procedures in the management of any requests for technical interventions of the equipment.

Cesare Quaranta devices are equipped with technical documents intended for the professional operators: user manuals and methods. They allow to simplify the operational functions and at the same time to ensure correct instructions to the area managers for a continuous and correct devices monitoring.

Each piece of equipment has been tested both from a functional point of view (reliability and performance) and from safety. It is carried out in full compliance with the standards set by the European Community and regularly submitted to strict compliance checks.

the EC standards:

- Safety. Our devices are EC marked and they comply with all the communitarian safety requirements.
- The high quality of our equipment is concrete and gives value to the aesthetic treatments even according to more and more demanding and selective customers.
- Efficacy. The treatment protocols are developed by professionals who have been operating for over 20 years in the aesthetic field. The aesthetic results are concrete and long-lasting, for the full consumer's satisfaction.

The company Quality System management aims at:

- Fulfilling and exceeding, if possible, the customers' expectations in terms of product quality
- Strengthening and improving its position on the market by a successful customer acquisition and long-term retention
- Stabilizing the manufacturing process in order to assure the highest functionality and reliability
- Keeping full attention to the cost saving policy



MADE IN ITALY PASSION

Decree-law n. 135 dated September the 25th, 2009, under article 16, comma 4 defines "made in Italy" as follows:

Whoever makes use of a sale indication presenting the product as completely made in Italy, like «100% made in Italy», «100% Italy», «all Italian», expressed in any language, or others to mislead consumer into believing that a product is entirely of Italian origin, that is, signs or figures leading to the same fallacious belief, outside the conditions foreseen in the comma 1 and comma 2, is punished, without prejudice to the different sanctions applicable on the base of the normative in force, with the sanctions foreseen under the article 517 of the Penal Code, increased by a third.

Cesare Quaranta production complies with the characteristics indicated by such a decree and it is 100% made in Italy.- Advertisements on radio and local newspapers.

- Organization of "group shopping" promotions

Our strategic view of the communication world makes us a central pillar of your strategy.


Assistance and warranty

Certainty to have specialized technicians at your complete disposal at any moment.

800/26.86.16 is the toll free number for the Client assistance. Cesare Quaranta is at your disposal to supply you with immediate services of technical intervention, and to answer all your requests or to get back to you with the information regarding the use of the devices.

Cesare Quaranta devices are guaranteed against defects in material and construction for a 12-month period.

Cesare Quaranta s.r.l. is synonym of high quality service towards any centre offering aesthetic and wellness treatments: the devices distinguish themselves by their compliance with the highest quality standards and by the extreme efficacy of their own methods.

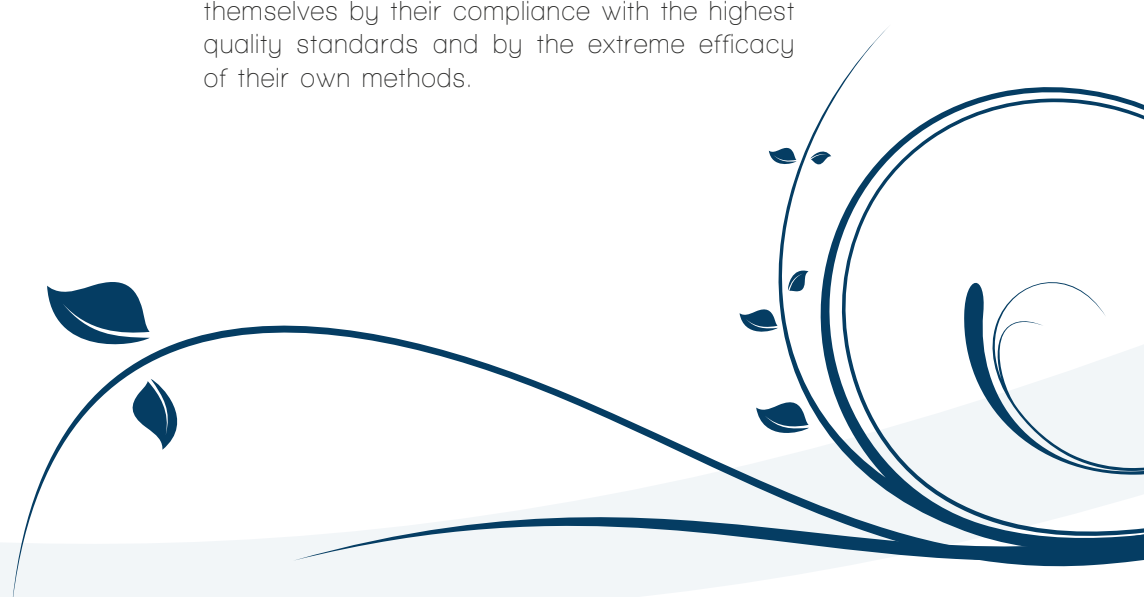

mya 6.0

mya 6.0

ADVANCED TECHNOLOGY UNLIMITED POSSIBILITIES

6 TECHNOLOGIES FOR RESULTS 6 TIMES MORE EFFECTIVE.

MYA 6.0 is the latest total body hand free device by Cesare Quaranta Srl.

Thanks to the action of six technologies that work in synergy, it fulfils the aesthetic wishes of beauty and well-being and it counteracts face and body imperfections quickly and effectively.

MYA 6.0 BOASTS THE 6 BEST
AESTHETIC TECHNOLOGIES ON A
SINGLE SUPPORT:

- ELECTROSTIMULATION
- ULTRASOUND
- LASER LLLT
- BIA TEST
- ELECTROPORATION
- PHOTOBIO-MODULATION

31 PROGRAMS, ENDLESS
POSSIBILITIES

- 2 programs for electrostimulation with face electrodes
- 11 programs for electrostimulation with body electrodes
- 13 programs for ultrasound / LLLT laser + electrostim on the same body plate
- program for the absorption of cosmetics and photobiomodulation
- 1 program for impedance test
- 1 chip card program
- 1 counter program
- 1 adjustments program

All programs can be customized according to the subject and the imperfection to be treated.

6 BENEFITS 6.0 TO RESHAPE
COMPLETELY THE BODY

- REMODELING
- SLIMMING
- DRAINAGE
- MUSCLE TONING
- TISSUE TONING
- TREATMENT OF CELLULITE IMPAIRMENTS


Simultaneously on the same plate to treat:

- LOCALIZED FAT;
- THE IMPERFECTION OF CELLULITE;
- MUSCULAR ATONY;
- THE LYMPHATIC STASIS;

LESS 6 CM IN THE AREA ABDOMINAL
protocol customize results in 15 sessions



MUSCLE TONING - TISSUE TONING
protocol customize results in 12 sessions

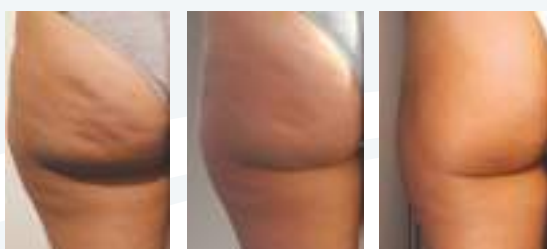

TREATMENT OF CELLULITE AND TONING
protocol customize results in 15 sessions



Power Supply: 220/250 V. single phase 50/60 Hz

Absorbed Power: 170 VA MAX

ELECTROSTIMULATION

Output Stage

Square wave - Kotz current Ultrasound/laser

Square Wave Frequency

1kHz modulated at 33/66/133/200 Hz

Kotz Wave Frequency

2500/4000/5000 Hz modulated at: 4/6/8/12/50/75/100 Hz

OUTPUT CURRENT

80 mA on a 500 Ω ohm

ULTRASOUND

Ultrasound Frequency: 3 MHz

Ultrasound Power: < 2.8 W/cm²

LASER

Wave Length: 635 nm

Potenza: 672 mW

COSMETIC ABSORPTION AND COLOURS

Output Stage: square wave

Square Wave Frequency: 1,6 kHz modulated at 300 mHz

Max Output Current: 70 mA load 500 ohm

Colours: Blue, white, yellow/orange, green

BIOIMPEDANCEMETRY

Current For BIA: 0.8 mA at 5 kHz, 50 kHz, 100 kHz

Number Of Output Channels:

8 right channels and 8 left channels for ultrasound, for ultrasound/laser, 4 right channels and 4 left channels for electrostimulation; they can be adjusted individually for each output, with analogue and digital indication on the display

Treatment Selection:

13 preset programs for traditional electrostimulation
13 preset programs x Ultrasound/laser and / or Electrostimulation
1 electroporation program
1 impedancemetry program
1 chip card program
1 counter program
1 setting program

Treatment Total Time:

Present to be set, from 5 to 60 minutes

Main Fuses: 2 x 2 A type T 5 x 20

Size: 420 x 1330 x 470 mm

Weight: 22 kg

Temperature Limits: 5° C to 40° C (when ON)

Classification:

Equipment for Aesthetics
CLASS I, BF TYPE

Standard Compliance:

2014/30/UE DIRECTIVE
2014/35/UE DIRECTIVE
2011/65/UE DIRECTIVE

Product: Made in Italy

The Manufacturer reserves the right to make all unannounced changes he considers necessary to improve the quality of the machine.

TECHNICAL DATA SHEET



6 TECHNOLOGIES ACTING TOGETHER ON THE SAME TIME INSIDE THE PLATE

ULTRASOUND

Ultrasound works by combining a localized thermal and micromechanical effect, thanks to which it acts on the imperfections of cellulite. In addition to massage and local heat production, device improves vascularization and molecular dissociation phenomena are obtained.

ELECTROSTIMULATION

Electrostimulation acts on face and body thanks to deep impulses, inducing a muscle and skin lifting that improves oxygenation and tissue revitalization. It promotes metabolic exchanges and the elimination of toxins.

The controlled emission of currents with different wave forms, modulated at frequencies ranging from 33 Hz to 5,000 Hz, allows the execution of customizable and effective aesthetic protocols.

LASER LLLT

LLLT laser sources with a wavelength of 635 nm from 672 mW in duty cycle with intermittent emission. Phases of activation and phases of relaxation are alternated to maximize the effects. LLLT laser technology has the ability to penetrate the skin barrier and to induce photochemical responses in the treated tissues, inducing metabolic reactions in the adipocytes creating a "cascade" reaction.

BIA TEST

The bioimpedance test evaluates the body composition of the customer. Thanks to the action of a sinusoidal current at three frequencies, it allows to carry out precise and reliable measurements of the percentages of lean mass, fat mass, total water, intracellular and extracellular liquids of the customer. This technology makes it possible to measure the condition of the subject before undergoing the treatment and to notice improvements throughout the cycle of sessions.

ELECTROPORATION

Electroporation allows to treat the imperfection spreading the cosmetic on the skin and favouring the total absorption of the active ingredients dissolved in it. The electroporator handpiece, by inducing small reversible structural changes in the stratum corneum, improves the permeability of the tissues.

PHOTOBIOMODULATION

Photobiomodulation is a method included in many cycles of well-being and it consists in irradiating the areas of the body with light of different colours to obtain an aesthetic advantage. It is combined with traditional beauty treatments to make the final result homogenous. In fact, the imperfection is considered as a disharmonious cells vibration of the body that can be rebalanced by exploiting the energy and, therefore, the vibration of one or more specific colors.



A UNIQUE PATENTED TECHNOLOGY WITH REAL EFFECTS



THE EVOLUTION OF THE PATENT WITH THE LLLT LASER

Strengthened by our experience in the study and production of ultrasound and electrostimulation plates - of which we hold the patent - we have incorporated LLLT laser technology on the same plates to emphasize and maximize the benefits and results obtained. LLLT LASER - LOW LEVEL LASER TREATMENT is the principle used by LLLT laser technologies that work in depth at a wavelength of 650 nm or 780 nm, directly on the hypodermis and fat.

ULTRASOUND AND LLLT LASER THE PERFECT SYNERGY FOR 6 TIMES MORE EVIDENT RESULTS

Ultrasound and LLLT laser work in synergy to emphasize the work promoted by electrostimulation on the target tissue according to the chosen program. Maximization and enhancement of results up to 6 times more in a single treatment. The powerful action of LLLT laser technology on fat accelerates weight loss by ensuring therecovery of tissue tone through the stimulation of collagen.

THE PLATES - COMPLETELY REDESIGNED

OVAL AND ERGONOMIC: new ultra-ergonomic shape, perfect for shock absorption.
BIGGER: to maximize the contact and action area.
NEW COLORS: white and blue for an elegant and innovative style.
OPTIMIZED FOR THE REFLECTION OF THE LLLT LASER: they guarantee total safety for the operator and the customer.

- + LLLT LASER
- + ELECTROSTIMULATION
- + HIGH FREQUENCY ULTRASOUND

BODY CHECK UP

SCIENTIFIC APPROACH

BIO IMPEDANCE TEST WITH THREE FREQUENCIES CURRENT

Thanks to the bio-impedentiometric test with which Mya is equipped it is possible to evaluate the client's body composition, by monitoring the condition before undertaking the treatment and by observing the improvements during all the session cycle.

The test allows carrying out, thanks to the action of a three-frequency sinusoidal current, precise and reliable measurements about the percentages of thin mass, fat mass, total intracellular and extracellular liquids of the subject.

Carried out before a session cycle, it allows elaborating a personalized work protocol aimed to reach the aimed aesthetic targets in the shortest possible time. After the test, the software with which Mya is equipped suggests the number of sessions and the ideal type of treatment for each subject.

Each one of the three frequencies used by the software acts like a fundamental function in the subject screening:

- **5 kHz:** contributes to evaluate the extracellular water;
- **50 kHz** is used for the thin mass measurement;
- **100 kHz:** allows the total water measurement.

The test is advised particularly for all the treatments aimed to the fat mass reduction, such as the localised fats and to fight the imperfection of cellulite sold out of the product contained in it.



ELECTROPORATION HANDPIECE BODY AND FACE

The handpiece focuses the action of chromo stimulation to four colors led and it combines the effectiveness of electroporation in delivering trans dermal active ingredients.

Handpiece treats the imperfections uniformly distributing the cosmetic on the skin and favoring the total absorption of active ingredients dissolved in it through electroporation action. It has the task of inducing small reversible structural changes in the stratum corneum, in this way it improves the permeability of tissues.

Mya E + can be equipped with a rich set of specific cosmetics to contrast the most common visage imperfections because they are studied to be conveyed effectively by exploiting the synergy between the chromium stimulation and electroporation:

- **Wrinkles:** Intensive for a visible long-lasting lifting effect;
- **Moisturizing:** Restore balance water nourishing the skin in depth.
- **Lightening:** It works on pigmentation restoring skin radiance.





LED MASK: A REVOLUTION IN THE FIELD OF PHOTOREJUVENATION.

THE SYNERGY OF TWO PRODUCTS: LED + NUCLEOTIDES SERUM

The face mask with red LED stimulates the biological response of the photoreceptors by inducing an effective biochemical reactions of tissues contrasting the signs of aging and sagging skin. The stimulation is athermic, painless, and it is suitable for any skin type.

The bio-stimulant action is exerted through the "LED anti-aging mask" which, thanks to 70 LED sources, distributes equally intensity and dose on the face surface. The result is a more toned, supple, youthful and healthy complexion in just few sessions.

Effects of ANTI -AGE treatment by Cesare Quaranta:

- It increases the production of new collagen by fibroblasts;
- It promotes the cell regeneration at epidermis level;
- It regenerates, hydrates and illuminates the skin;
- It helps the skin to better resist to external agents;
- It reduces the ageing signs;
- It brings firmness and suppleness to tissues.

Effects of REPAIR treatment by Cesare Quaranta

- It reduces inflammation and skin sensitivity;
- It favours the cell regeneration at epidermis level;
- It moisturises and illuminates the skin;
- It helps the skin to better resist to external agents;
- It reduces the acne inflammation.

LED: Visible red or blue light

Dimensions: 300 x 300 x 380

Numbers of LED sources: 70

Wave Length: 645 nm

Absorbed power: 4W

CLASS II

Compliance to

EN61347-1, EN61347-2-13, EN55015, EN61000-3-2,
EN 61000-3-3, EN 61547, EN62493-2010,
EN1194-2012



Hand LED Mask

HIGH-TECH COSMETICS FOR YOUR SKIN

activa⁴⁰
Anti-Age Serum

activa⁴⁰
Repairing serum



NUCLEOTIDES DNA REGENERATIVE EFFECT

The core of all serum formulas of activa⁴⁰ range are the nucleotides of natural origin. They are able to moisturise, protect and stimulate the skin renewal.

Nucleotides are natural ingredients which are part of the nucleic acids structure (DNA and RNA).

They participate to molecular processes determining the genetic information coding and transfer to the cells. This characteristics make them **extremely useful during the cell regeneration because they accelerate the natural repair processes of our body.**

All activa⁴⁰ serums have been conceived to favour effects on cells. Thanks to methyl-propanediol, the transdermic penetration of active principles is improved. **The serums are free of preservatives, parabens and perfume in order to be safe and suitable for all skin types, even for the most fragile and sensitive areas.**


ANTI AGE HAND LED MASK

Hands, as face and décolleté, are much exposed to ageing caused by external agents, jumps in temperature, pollution effects.

The anti-age treatment is ideal also on hands, taking advantage of the nourishing effect of Activa 40 Anti-age serum rich in polynucleotide's and combined with specific HAND LED mask action of light energy at 645 nm. As well as the face and neck anti-age treatment, even the hand treatment starts with the application of a tnt wrap soaked in the Activa40 Anti-ag serum and then proceeds to exposure of hands to red LED light.



BEFORE

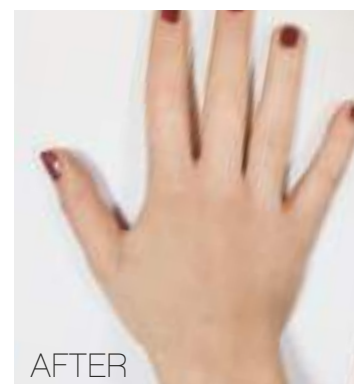

AFTER

LED: Visible red light

Dimensions: 250 x 205 x 95

LED numbers: 25

Wave Length: 645 nm

CLASS II

Compliance to

EN61347-1, EN61347-2-13, EN55015, EN61000-3-2,
EN 61000-3-3, EN 61547, EN62493-2010,

HPL

HIGH POWER FRACTIONAL QUANTUM ENERGY

HIGH POWER FRACTIONAL QUANTUM ENERGY

HPL is a photobiomodulation device that works through the utilization of non-ionizing electromagnetic energy to trigger photochemical changes within cellular structures that are receptive to photons.

HOW IT WORKS:

The frequency-modulated with high-intensity, through 3 handpieces with three different wavelengths, irradiate the different tissues of the body with energy in order to obtain evident and lasting results. the original three wavelength high power system that through 3 handpieces irradiate the different tissues of the body with energy in order to obtain evident and lasting results.

HPL energizes cells and reorganizes tissues

- Erythema
- Dermatitis
- Wrinkles
- Scars
- Laxity and wrinkles of the neck
- Shingles

HPL regenerates the skin for a compact and youthful appearance

- Cellulite lower limbs, abdominal and upper limbs
- Periocular (crow's feet) and prior (mouth contour) wrinkles
- Skin relaxation and muscle atony
- Localized adiposity
- Circulatory deficits and lymphoedema
- Skin blemishes on the face and body

WAVELENGTH	BLUE 460 nm
WAVELENGTH	RED 670 nm
WAVELENGTH	infraRED 870 nm
ELECTRICAL SUPPLY VOLTAGE OF THE DEVICE	18V



BEFORE

AFTER



AIRYe+®

ENDODERMIC MASSAGE: IT REACTIVATES THE CONNECTIVE TISSUE FUNCTION

Airy is treating cellulite and fat imperfections, is toning tissues and tightening skin thanks to 3 technologies combined together and working simultaneously: endodermic massage (by suction), resistive radiofrequency and LLLT.

Targeting arms, legs, hips, abdomen, face and décolleté, by different size hand-pieces, **Airy** is suitable in a totally safe way to all body areas. The treatment has no counter-indications, is nice to feel and effects are visible since the first application.

Airy applies at the same time the effect of:

- Endodermic massage (by suction);
- Resistive Radiofrequency;
- Laser light (LLLT).

Airy on body:

- reduces fat;
- smooths cellulite;
- tones and firms skin.

Airy on face:

- reduces the fine wrinkles;
- redefines the facial contour;
- clarify and lift up skin.

RESHAPING IS BRANDED CESARE QUARANTA

A scientific approach to the aesthetic imperfection is necessary when you look for the best results in body remodelling. You start with the physical analysis of the subject and gather all the important information allowing to define the most efficient slimming protocol.

This is the reason why, besides the aesthetic treatments, the Cesare Quaranta method includes:

- A clear and accurate body check up
- A food programme
- A team of consultants at your disposal
- Continuous education programme

To lose inches, to remove fat and treat cellulite: it doesn't matter what is the target.

The customer can achieve it through a completely customized programme, supported by the knowledge of our highly professional staff. Every treatment comes out from research over the years and guarantees fast and durable results, respecting the natural physiological response of the body.

TECHNICAL DATA SHEET

Power Supply: 230 V, 50/60 Hz

Power absorption: 250 VA

VACUUM

Vacuum pump

Final rated pressure: 150 mBar (abs)

Suction rated capacity 3-3,6 m³/h

Operation modes Continuous suction | Intermittent suction. Possible modulation of suction ("vibration" mode)

RADIOFREQUENCY

Type: Resistive

Frequency: 500 kHz

Power: <25 W

RED LIGHT

Number of light sources: 4

Wave length: 635 nm

Power IN Diode: 5 mW

Total power: 20 mW

Power density: <10 mW/cm²

Class: III R

Mains fuses: 3,15 A - Ritardato - 5 x 20

Dimensions: 1350 x 450 x 440 mm

Weight: 25 kg

Classification: I, type B

Compliance: Directive 2014/30/CE | Directive 2014/35/

Operating temperature: 12° - 30° C

Relative humidity: 30 - 75%, without condensation



ENDODERMIC MASSAGE: IT REACTIVATES THE CONNECTIVE TISSUE FUNCTION

The main action of the endodermic massage is to move stored fat by suction. Then, the fat is released in the cutaneous and subcutaneous tissues. The operation cycle is split in 2 phases, alternating a deep and intense work to a soft and relaxing one.

The endodermic massage impacts tissues locally, re-activates the cells activity and stimulates the micro-circulation, increasing the tissue oxygenation. It also promotes the relaxation of collagen fibre.

Laser LLT slimming action

Its action targets mainly fat tissues. The laser light is able to go through the cutaneous barrier and solicit photochemical reactions in the target tissue, activating then the lipolysis of adipocytes and provoking enchainned reactions.

Laser light increases the gradient of the cytochrome membrane, increases ATP exchanges and their synthesis and converts triglycerides in fat acids and glycerol which can then pass through membrane transitional pores and be released.

Resistive Radiofrequency: anti-age lifting

Resistive radiofrequency helps in a considerable way to improve the skin texture, thanks to its direct action on collagen fibres which are the skin structure.

Its effect is immediate and progressive: improvement of derma is visible since the first treatment and goes on in the following days as the fibroblasts keeps on producing new collagen.

The hyperthermia led by radiofrequency also reactivates vascularisation which is important for a good detoxification.



AIRYe+

SCULPTING ON BODY GENTLE ON FACE



Two hand-pieces of different size allow to safely treat all body areas. The results in reshaping are extraordinary and the reduction of cellulite and fat imperfections is considerable.

Liposculpting and cellulite smoothing

The endodermic massage promotes the mobilisation of adipocytes from the connective tissue. The fat cells are submitted to 4 laser light at the same time coming from 4 LLLT sources at 635 nm.

This increases the permeability of their membrane thus stimulating the fat release.

The heat provoked by radiofrequency improves the blood circulation promoting both drainage of stored liquids, responsible for cellulite appearance, and release of fat.

Improvement of skin texture

The endodermic massage redensifies the connective tissue and relaxes the collagen fibres. Thanks to radiofrequency, the production of new collagen and elastin by fibroblasts is reactivated.

The thermal effect improves the microcirculation of tissues and their oxygenation, thus skin texture regains clarity, tone and firmness.

The 4 interchangeable heads of the handpiece allow a targeted action on a face, neck and décolleté areas. The endodermic massage on face reactivates and restores the connective tissue and firm the skin stimulating the production of new collagen and elastin and increasing the microcirculation and oxygenation of tissues.

An immediate redefinition of the facial contours and an important reduction of the fine wrinkles are visible since the first session.

Especially where the skin appeared fragile and dull, the correct elasticity and radiance are restored.

Airy on face:

- reduces the fine wrinkles
- defines the facial contour
- restores skin radiance and tone



*But thou, contracted to thine own bright eyes,
Feed'st thy light's flame with self-substantial fuel,*

Giugno 2021



CESARE
QUARANTA

CESARE QUARANTA s.r.l. Unipersonale

Via Legnanino, 5 - 10023 Chieri (TO) Italy - +39 0119472132
www.cesarequaranta.com - e-mail: info@cesarequaranta.it